




Internal precision fit for safe, comfortable retention.  
 Made with hygienic, durable Clear 450™

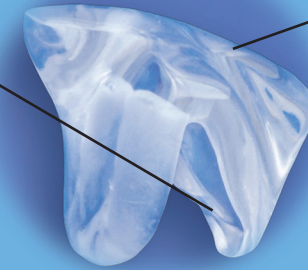

Discluding Element (DE)



**Indications:**


- Narrow lower arch
- Lower veneers oppose natural maxillary teeth
- Class III occlusion
- Lower uneven incisal edges
- Missing lower incisors
- Long protrusive > 8mm

**Nighttime Lower**


**First choice for function and comfort.**

**Contraindications:**

- Overbite > 50%
- Short lower clinical length lacking retention
- Overjet > 3mm

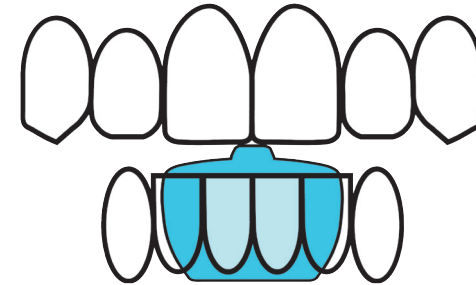


Front View

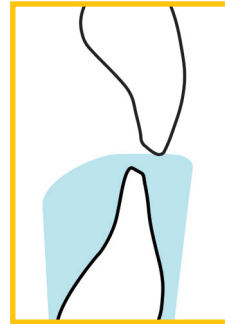

Side View

**Nighttime Upper**

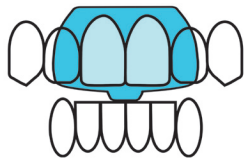


**Indications:**


- Overbite > 50%
- Uneven maxillary incisors
- Maxillary veneers
- Mobile upper incisors
- Lower incisors will not provide adequate retention.
- Overjet > 3mm

**Contraindications:**

- Narrow lower arch
- Lower veneers
- Class III Bite
- Uneven lower incisors

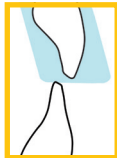

**Modified Z Upper Only**

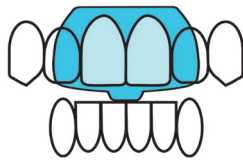

This design's sloped lingual DE allows for a reduced VDO for patients with a deep bite.

**Indications:**


- Deep Bite

**Contraindications:**

- Uneven lower incisors.
- Missing lower incisors
- Narrow lower arch.
- Lower veneers.



**Daytime NTI**

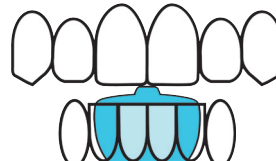

**Indications:**


- General Upper/Lower Indications plus:
- Migraineur or severe headache patient needing daytime therapy for muscle relaxation and pain relief.
  - A "habit" appliance for daytime bruxer or clencher.

**Contraindications:**

- May adversely affect speech or appearance for some patients.



**Universal Therapy Set**

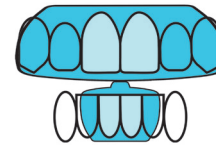

Flat plane to canines used in conjunction with Nighttime NTI as part of the Universal Therapy Set.

**Indications:**


- Incisal height difference on both arches
- Patient divots or breaks NTI-tss Plus
- Veneers on both arches
- Missing an incisor on both arches

**Contraindications:**

- If concerned about additional vertical opening, the Universal Therapy Set may not be recommended.



**Extended Coverage**

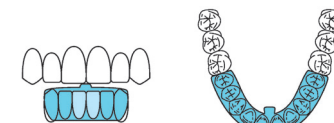


Standard Design covers the centrals & laterals, but the extended design allows for any combination of coverage.

**Indications:**


- Fixed lingual retainer
- Increased retention for short clinical length
- Added strength

**Contraindications:**

- May result in additional vertical opening
- Extreme gag reflex

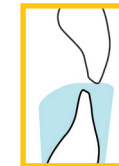

**Soft-Lined NTI**

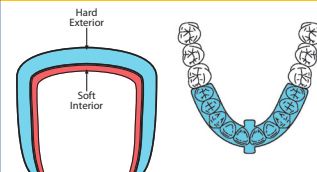

Hard exterior with soft interior. Extends to 2nd bicuspids for optimal comfort & sufficient retention.

**Indications:**

- Veneers
- Slight Mobility
- Maximum patient comfort

**Contraindications:**

- Strong gag reflex
- May result in additional vertical opening.

